





aerolight-storage bracket

Instructions for use

Before Use


- Remove all packaging and ensure the product has not been damaged in transit. Retain packaging.
- Carefully read the instructions and warning texts before using the product.
- Keep instructions for reference

To Install

- Fix storage brackets to wall or other suitable flat surface at same level with suitable fixings
- Fix padlock loop centrally over stored ramp position with suitable fixings

Use

- Place ramp in brackets
- Secure with padlock around padlock loop and both handles of ramp

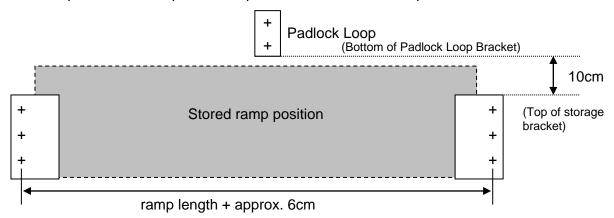

Maintenance

- If the product/components become damaged in any way they should not be used (contact supplier/manufacturer).
- The product may be cleaned with normal cleaning products.

Warranty

- The product carries a comprehensive 2 year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- Any modification/repair other than by manufacturer will void the warranty.



RampCentre is a registered trademark of Enable Access, LU7 4UH. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 09 Dec 2020

Enable Access, Unit R, Cherrycourt Way, Leighton Buzzard, LU7 4UH, UK. Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

For technical support or details on your nearest stockist please call 020 8275 0375















aerolight-storage bracket

Instructions for use

Product code: ASB

Other Accessories Available (contact supplier/manufacturer to order)

Code:

Security alarm **AMSA** Warning cone **AWC AWA** Wall strap kit **AWS** Wheelchair access sign

Other products available include lightweight platforms, ramp joining supports and handrail kits.

RampCentre is a registered trademark of Enable Access, LU7 4UH. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 09 Dec 2020

Enable Access, Unit R, Cherrycourt Way, Leighton Buzzard, LU7 4UH, UK. Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

For technical support or details on your nearest stockist please call 020 8275 0375





