

aerolight-upright storage bracket

Instructions for use

Product code: ASBV


Contents:

- 1 Base Unit (black)
- 1 Top (lockable) Unit (black)

1. Fix base firm and level to suitable surfaces using (4) fixing holes in bottom and back
2. Mark and drill the (4) fixing holes for the top (lockable) unit precisely above the base.
Dimension from top lip of base to bottom of lockable top unit (with door open) =
Ramp length – 12cm
i.e.: 138cm for AX15 (150cm) Ramp
3. Securely fix the top unit to the wall.

Please Note:

- A. The door of the top (lockable) unit should fit into the hollow of the 'S' shape lip of the AeroLight ramp (see picture below)

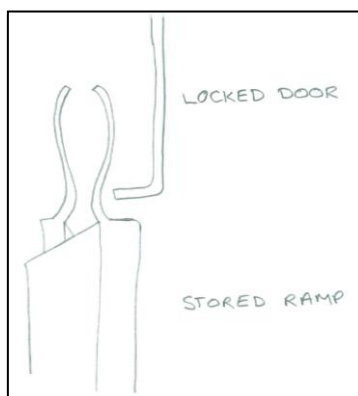

- B. Ensure ramp can be easily placed in open storage kit and not removable from the locked unit. If necessary adjust.
- C. This storage kit is designed specifically for use with AeroLight Xtra wheelchair access ramps

Maintenance

- If the product/components become damaged in any way they should not be used (contact supplier/manufacture).
- The product may be cleaned with normal cleaning products.

RampCentre is a registered trademark of Enable Access, LU7 4UH. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 09 Dec 2020



Enable Access, Unit R, Cherrycourt Way, Leighton Buzzard, LU7 4UH, UK.



Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

For technical support or details on your nearest stockist please call 020 8275 0375

Enable Access

Unit R | Cherrycourt Way | Leighton Buzzard | LU7 4UH | UK

T: +44 (0)20 8275 0375 **E:** sales@enable-access.com

W: www.enable-access.com



Celebrating over
20 YEARS
of better access
for better living



aerolight-upright storage bracket

Instructions for use

Product code: ASBV

Warranty

- The product carries a comprehensive 2 year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- Any modification/repair other than by manufacturer will void the warranty.



Other Accessories Available (contact supplier/manufacturer to order)

RampCentre is a registered trademark of Enable Access, LU7 4UH. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 09 Dec 2020



Enable Access, Unit R, Cherrycourt Way, Leighton Buzzard, LU7 4UH, UK.

Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

For technical support or details on your nearest stockist please call 020 8275 0375

Enable Access

Unit R | Cherrycourt Way | Leighton Buzzard | LU7 4UH | UK

T: +44 (0)20 8275 0375 **E:** sales@enable-access.com

W: www.enable-access.com



VAT Reg. No. GB 894 0149 09. Enable Access Ltd. Registered in England and Wales No. 10945974. Registered Office: As Above.