



Celebrating over  
**20 YEARS**  
of better access  
for better living


**Enable**  **Access**

**Metro**™

Mobility Products

## Instructions for use

Product codes: MBR4, MBR6, MBR8, MBR10

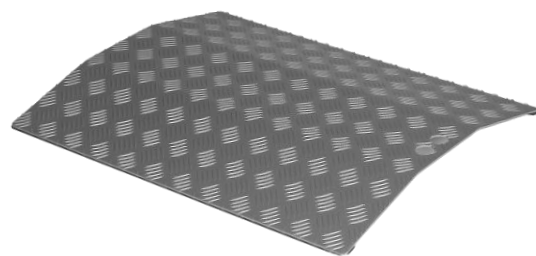

**Intended Purpose:** These ramps are intended to be used as a wheelchair accessory for persons with disabilities to navigate areas of different heights.

### Before use

- Remove and retain all packaging
- Ensure the ramps have not been damaged in transit or storage.
- Carefully read the instructions and warning texts before using ramps. Keep instructions for reference.
- Ensure the surface at both levels is firm, stable and able to bear the intended load.
- The capacity of the ramp is 250kg (39.4st) assuming the ramp rests on the threshold.

### To Deploy

1. Position the ramp to bridge over the threshold (pic 1).
2. Ensure the ramp does not become dislodged in use. Check ramp is firmly located, and stable each time it is used. Take care when using the ramp, so that no part of the wheelchair catches and moves the ramp.

### Maintenance

- If the ramp becomes damaged or slippery in any way it should not be used (contact supplier/manufacturer).
- The ramps may be cleaned with normal cleaning products and left to dry before use.
- Maximum recommended cleaning temperature 50°C; please enquire for further details on use with industrial cleaning machines.

### Recycling

- For disposal of this product, please liaise with a metals recycling contractor.
- All the materials in the ramp are 100% recyclable. Detach all rubber detail and send the aluminium parts to metal recycling.

### Warranty

- The ramp carries a comprehensive 2-year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- The ramps are not designed for permanent outside use and must not be fixed down.
- Any modification/repair other than by manufacturer will void the warranty.



Metro Mobility is a registered trademark of Enable Access, LU7 4UH. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 15 August 2022



Enable Access, Unit R, Cherrycourt Way, Leighton Buzzard, LU7 4UH, UK.



Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

**For technical support or details on your nearest stockist please call 020 8275 0375**