

utilityramp

Assembly and Use Instructions

Product codes: UR9, UR12, UR15, UR18, UR21, UR24,
URW9, URW12, URW15, URW18, URW21, URW24

Intended Purpose: These ramps are intended to be used as a wheelchair accessory for persons with disabilities to navigate areas of different heights.

Before use

- Remove all packaging and ensure the ramp has not been damaged in transit. Retain packaging.
- Carefully read the instructions and warning texts before using the ramps
- Keep instructions for reference
- On the ramp you will find a label that shows the maximum permitted load for the ramp.

Deploy/Use

- Carry/move the ramp using the handle and wheels (where fitted – see table below).
- Ensure the ground/surface at both upper and lower levels is firm and stable.
- Check ramp is long enough to cover the step/rise. The ramp is designed for use between 1:4 and 1:20 gradient (dependent on safe application and use requirements).
- Place the ramp in position with the lip on the upper level, central to the chosen route.
- Ensure the ramps do not become dislodged in use. Check ramp is firmly located and stable each time it is used.

Maintenance

- If the ramp becomes damaged or slippery in any way they should not be used (contact supplier/manufacture).
- The ramp may be cleaned with normal cleaning products and left to dry before use.

Warranty

- The ramp carries a comprehensive 2 year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- These ramps are not designed for permanent outside use and must not be fixed down.
- Any modification/repair other than by manufacturer will void the warranty.

Ramps will not be repaired/replaced if the serial number plaque has been removed/replaced

Key product information



Code	Length	Width	Capacity	Weight
UR9*	90cm (3ft)	76cm (30ins)	400kg (63st)	9kg (20lbs)
UR12*	120cm (4ft)	76cm (30ins)	400kg (63st)	12.5kg (28lbs)
UR15	150cm (5ft)	76cm (30ins)	400kg (63st)	15.5kg (34lbs)
UR18	180cm (6ft)	76cm (30ins)	400kg (63st)	19kg (42lbs)
UR21	210cm (7ft)	76cm (30ins)	350kg (55st)	22.5kg (50lbs)
UR24	240cm (8ft)	76cm (30ins)	300kg (47st)	25.5kg (56lbs)
URW9*	90cm (3ft)	90cm (35ins)	400kg (63st)	10.5kg (23lbs)
URW12*	120cm (4ft)	90cm (35ins)	400kg (63st)	14.5kg (32lbs)
URW15	150cm (5ft)	90cm (35ins)	400kg (63st)	18.5kg (41lbs)
URW18	180cm (6ft)	90cm (35ins)	400kg (63st)	22kg (49lbs)
URW21	210cm (7ft)	90cm (35ins)	350kg (55st)	26kg (57lbs)
URW24	240cm (8ft)	90cm (35ins)	300kg (47st)	30kg (66lbs)

* No wheels. Kerb height 7cm (3ins)

utilityramp

Assembly and Use Instructions

Product codes: UR9, UR12, UR15, UR18, UR21, UR24,
URW9, URW12, URW15, URW18, URW21, URW24

Utilityramp handle kit and Utilityramp wheel kit fitting instructions:

- All UR and URW ramps use the Utilityramp Handle Kit (URHK)
- UR and URW ramps 150cm and longer also use the Utilityramp Wheel Kit (URWK)



Utilityramp handle kit (URHK) contents

- HD Ramp Peg (UR) x2
- M5 x 16 Coach Bolt x2
- Handle x1
- M5 Flange Nuts x2
- M8 x 55 Coach Bolt x2
- M8 Flange Nuts x2
- 1/4" Tommy Bar
- 7/8" Tube Spanner
- 13mm Tube Spanner



Utilityramp wheel kit (URWK) contents

- M6 x 20mm Coach Bolt x8
- M6 Flange nut x8
- 60mm Wheel Fixed Plate Castor x2
- 10mm Tube Spanner

To fit ramp pegs:

1. Locate the ramp pegs to the underside of the ramp at the threshold end on the 2nd set of holes from the end of the ramp.
2. Fix with M8 x 50 coach bolts x2 and M8 flange nyloc nuts x 2 using T bar and socket provided in kit (a screwdriver will also be required)

To fit handle:

1. Locate 2 x pre-drilled holes on the side of the ramp
2. Fit handle using M5 x 16mm coach bolt x2 and M5 Flange nut x2 (fixing tools supplied)

To fit castors:

1. Locate the castors to two sets of 4x holes on the side of the ramp
2. Fix with M5 coach bolts x8 and M5 flange nut x8 (fixing tools supplied)

